



Salzmann
MEDICO

SALZMANN AG
Salzmann MEDICO
Rorschacher Strasse 304
CH-9016 St.Gallen
Tel. +41 (0) 71 282 12 12
Fax +41 (0) 71 282 12 10
medico.sg@salzmann-group.ch
www.salzmann-group.ch

www.venosan.com



REF 995 107E-12/10 asi

VENOSAN® is a registered trademark of SALZMANN AG, St.Gallen, Switzerland



VENOSAN®

Measuring instructions Compression stockings



finest
comfort

Measuring for VENOSAN® medical compression stockings and pantyhose



📄: see www.venosan.com ➔ For EUROPE ➔ SUPPORT ➔

Videos how to measure for Standard size and custom made stockings

When are the measurements taken?

Measurements should be taken in the **morning** for patients whose legs have a pronounced tendency to swell.

How are the measurements taken?


- **Standing** upright using a measuring tape.
- Take the **circumference** and **length measurements** from the foot up to the thigh at the measuring points shown below.
- Each **circumference** has its own **length**.
- Measure the lengths with the leg held straight.
- **Both** legs must be measured separately.

Where are the measurements taken?

The relevant positions are shown in the figures below.


Circumferences:

Below knee

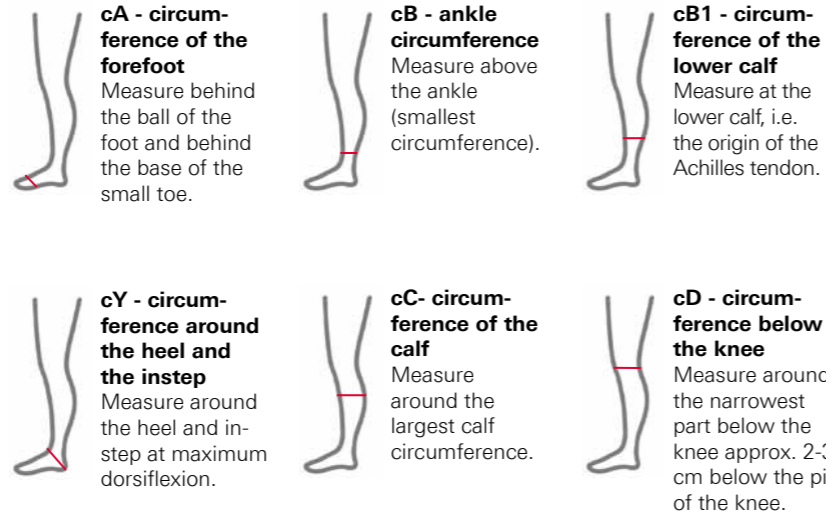


Lengths:

Below knee

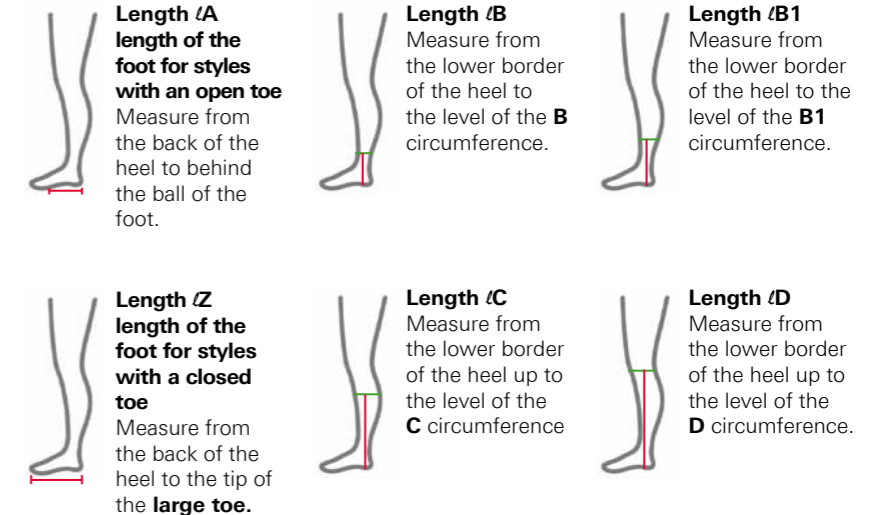


Thigh length stockings

Measurements in addition to the basic measurements for knee socks

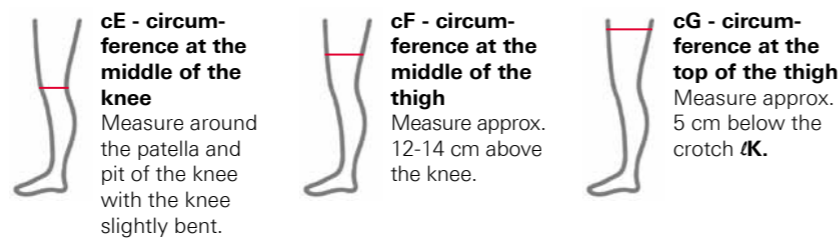


Thigh length stockings

Measurements in addition to the basic measurements for knee socks

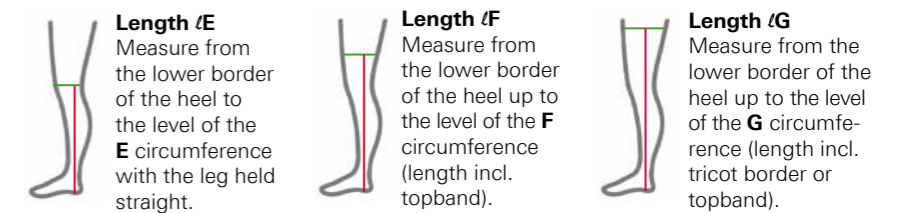

Pantyhose

Measurements in addition to the basic measurements for knee and thigh length stockings

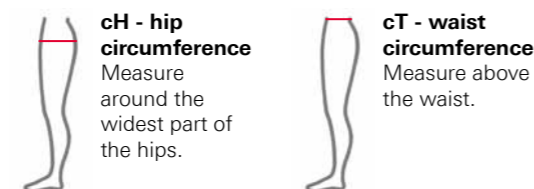

Pantyhose

Measurements in addition to the basic measurements for knee and thigh length stockings

